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11.01.2016 /dt.-engl.

Megaseal Universal properties and advantages:

- **easy and fast application**
- **high fitting accuracy**
- **high scratchable resistance**
- **high gloss oversize**
- **cureable in all dental light units**

1. Product description

Megaseal Universal is a light curing lacquer for the coating of resin parts of acrylic dentures as well for temporaries in C+B technique. A high abrasion-resistant and high gloss polished surface can be produced.

2. Application

Megaseal Universal can be applied with a brush directly onto the oversize of the denture or temporary. For safe adhesion, the surface should be free of grease and dry.

The layer should be thin. If a thicker layer is required, the application should be in 2 steps with each separate curing process.

The curing process should be effected in a light-polymerization unit with UV- or Bluelight spectrum (ea. Megalight) with a wavelength of 320-500 nm. Based on the intensity of the unit, the curing time should be 3-5 minutes.

3. Technical data / Important advice

Attention! Megaseal Universal contains Methylmetacrylate!

It is flammable liquid. Keep away from fire or heat.

Do not inhale and do not get in contact with skin or eyes.

Contains: Methylmethacrylate, Polyacrylate, Photoinitiator, Stabilisator

4. Shelf life

2 years

5. Package and delivery units

20 ml bottle with brush insert

30 ml bottle with drop insert

Art.-Nr. 2VLACK-UV2

Art.-Nr. 2VLACK-UV

6. Date information prepared

11.01.2016 /dt.-engl.